

## Seal Protect® Cast Cover

The SealProtect shower cover offers safe protection for both (plaster) bandages and the skin during showering. The shower cover is reusable and easy to apply. Thanks to the strong plastic ring and the sturdy silicone material in the ring the cast cover always closes watertight. The durable taffeta vinyl of which the shower cover is made ensures that the cover lasts for a long time.

### ADULTS

Art.nr.	Description
TH 18210013	Sealprotect Adult Short Arm Ø 15 cm, Length 51 cm
TH 18210113	Sealprotect Adult Long Arm Ø 19 cm, Length 66 cm
TH 18210213	Sealprotect Adult Ankle Ø 19 cm, Length 32 cm
TH 18210313	Sealprotect Adult Short Leg Ø 19 cm, Length 61 cm
TH 18210413	Sealprotect Adult Hand Ø 15 cm, Length 32 cm
TH 18210513	Sealprotect Adult Long Leg Ø 25 cm, Length 102 cm
TH 18211313	Sealprotect Adult Short Leg Extra Ø 25 cm, Length 57 cm

### CHILDREN

Art.nr.	Description
TH 18212413	Sealprotect Pediatric Arm S Ø 15 cm, Length 32 cm
TH 18212013	Sealprotect Pediatric Arm M/L Ø 15 cm, Length 50 cm
TH 18212113	Sealprotect Pediatric Leg M Ø 19 cm, Length 44 cm
TH 18212213	Sealprotect Pediatric Leg S Ø 15 cm, Length 23 cm
TH 18212313	Sealprotect Pediatric Leg L Ø 19 cm, Length 61 cm



### Measurement instructions:

- Hand / arm: measure from the fingertip of the middle finger to the end of the cast, then add 4 cm to determine the correct length of the cover.
- Ankle / foot: to determine the correct size, measure from the heel to the end of the cast and then add 4 cm.
- With a large or wide (plaster) bandage, also pay attention to the diameter of the ring, in that case choose a shower cover in a larger size with a wider opening.

### Instructions for use:

1. Remove all sharp objects, if necessary first apply a sock or stocking over the (plaster) bandage.
2. Hold the plastic ring firmly and make a large opening with your hands, then pull the cover over the (plaster) bandage.
3. Make sure the cover closes at least 3 cm above the (plaster) bandage for the most complete protection.
4. Carefully push the plastic ring slightly backwards so that the edge of the opening points upwards.
5. Dry before removing the shower cover.

### General information:

- The covers are only suitable for showering and not for bathing or recreational use such as swimming.
- Check the cover for damage or cracks before each use to prevent damage to the (plaster) bandage.
- Allow the cover to dry thoroughly before storing.
- The cover is Latex-free.

### Safety Warning:

- Use with care, the cover may become slippery when in contact with water.
- Adult supervision is recommended when using a shower cover on children.
- Not suitable for recreational use.
- The plastic ring is not designed for a heavy weight to apply.
- Towel dry. Caution! Do not dry on the heater.