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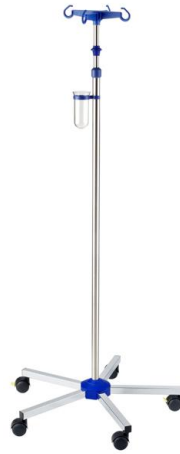
## 1. General Information

### Information relating to the instructions for use

These instructions for use should enable you to use the IV-stand safely and properly. Please read these instructions through carefully before using the IV-stand for the first time.

Pay particular attention to the instructions. If you give the IV-stand to a third party, please ensure that you also pass on the instructions for use at the same time.

Non-compliance with these instructions can lead to serious injury or may damage the IV-stand.

### Correct use

The IV-stand series I-N is an IV-stand for gravity infusions. The IV-stand is **only** for the support of infusion bags, infusion bottles and up to 2 pumps.

### Liability and warranty

All of the information and instructions provided are given to the best of our knowledge and are based on our experience and knowledge to date. We reserve the right to make technical changes essential for subsequent product development.

**provita assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.**

### Symbols in the instructions for use

Important instructions in these instructions are highlighted with symbols, as shown below:

**⚠️ WARNING!**

**This symbol refers to safety instructions. Failure to comply with these could result in injury or even death.**

**⚠️ ATTENTION!**

**This refers to risks that may result in damage to the IV-Pole.**

**👉 NOTE!**

Contains tips and other useful information listed in these instructions for use.

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## 2. Safety

- It is absolutely necessary when setting up, handling and using the device that these instructions are followed precisely.
- **This device is not MRI compatible.**
- This product should only be used under proper supervision from a healthcare professional.
- The allowed maximum weight per bottle-hook is 2 kg / 4,4 lbs.
- Pumps can be attached. But only on the **outer-tube**. The max. load capacity (pump and all infusion bags) should not exceed 9,5 kg / 21 lbs (with weighted base 11 kg / 24,5 lbs). **The clamp of the pump should not damage the tube!**
- Do not mount equipment that places the load off center of the pole.
- Assurance against tipping (standing assurance) is guaranteed for up to 5° inclination. Do not use or dispose the IV-Stand on inclined support!
- Use caution if IV pole is fully loaded. Product may tip.
- When moving, make sure that the roller-brakes have been unlocked. The IV-Stand has to be moved by hand only! Rolling across obstacles, soft flooring and slants must be done with particular care. **The inner tube has to be at shortest / min. position.**
- Do not stand on or ride this device.
- Use caution around stairwells.
-  **Attention! Do not open the screw for height adjustment without holding the inner tube because the load might slide down uncontrolled.**
- The height adjustment screw has to be tightened only as much as the load is held tight secure. Overwinding the screw might damage the IV-Stand.
- Infusion accessories may only be used on IV-Stands manufactured expressly for that purpose, as otherwise safety from tipping is no longer guaranteed.
- IV-Stands not in use are always to be locked with the lock rollers.
- Brakes may not stop IV pole on all surfaces.
- The electrically conductive rollers serve only to divert static charges.
- The person who is selling the product, is only responsible for the safety and proper functioning of the IV-Stands when the directions for assembly and use have been followed in all points and only when original accessories are used.
- Failure to adhere to these warnings can result in severe injury.

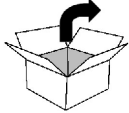
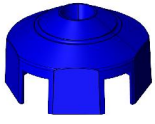
## 3. Installation

### Unpacking

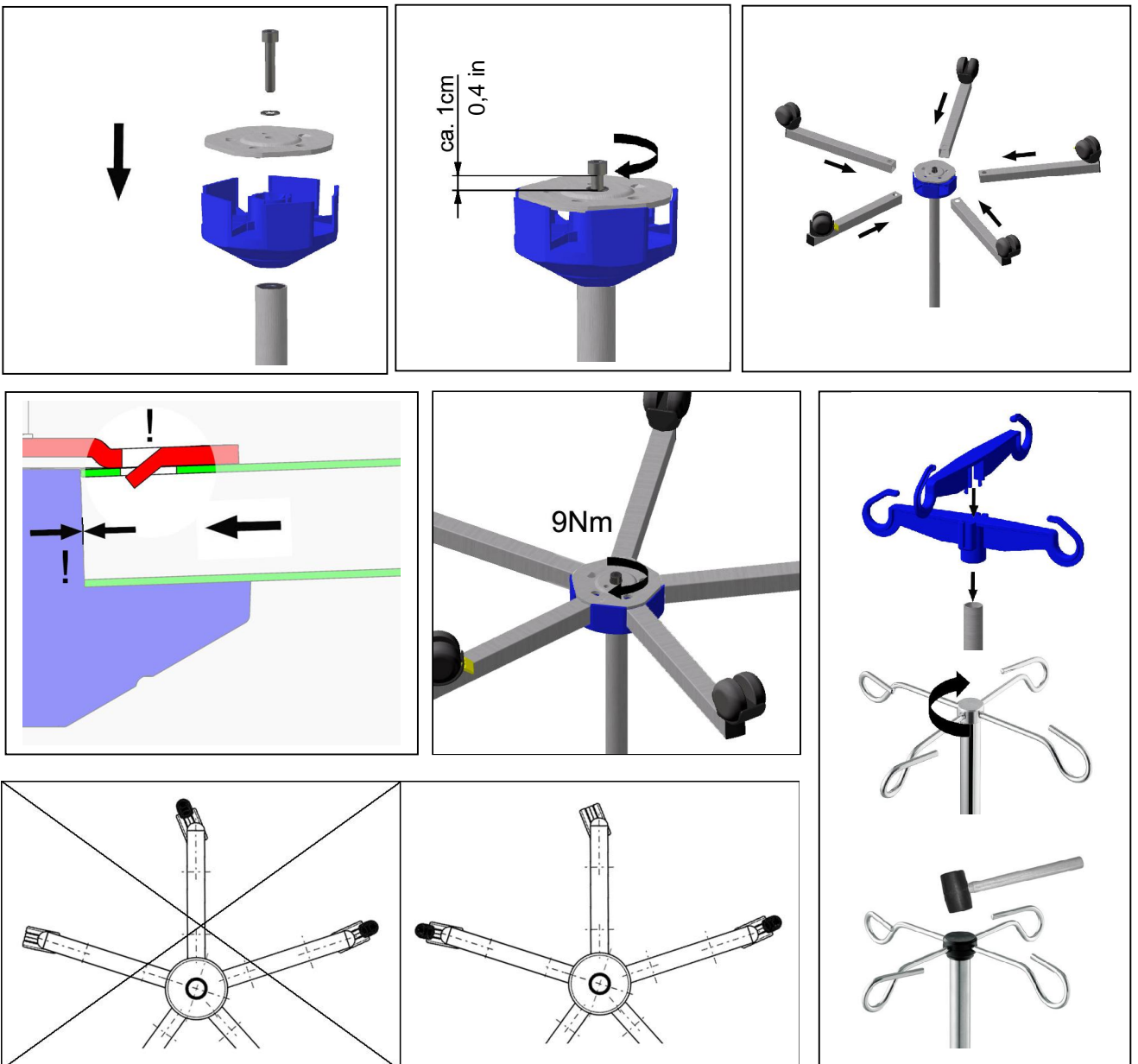
Unpack the IV-Stand and all of the accessories and check that the contents are intact.

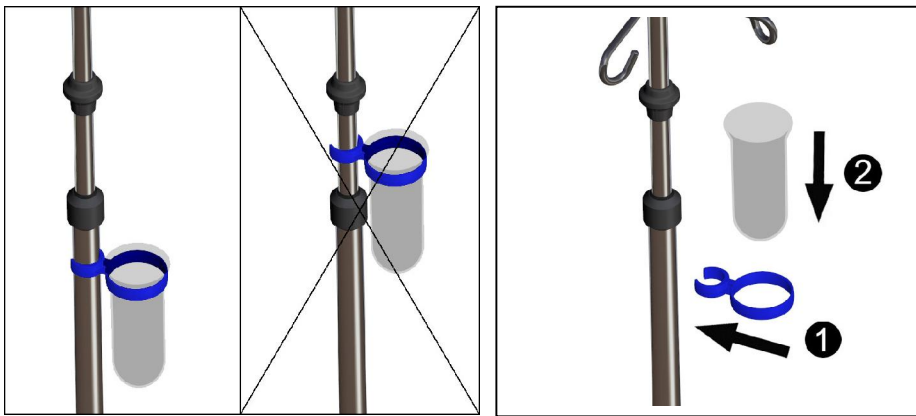
**The content of the packaging can vary in case of customized/special IV-Stand models.**

Dispose of any packaging materials that are no longer required in accordance with the current legislation in your country. Please contact your retailer immediately if you observe any damage caused during transport.

	 1x		
3x 	2x 		
1x 	1x 	1x 	
1x 	1x 	1x 	1x 

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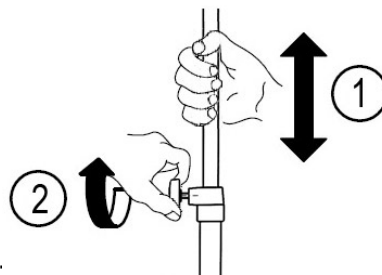
#### 4. Operating Instructions

1.) For adjustment of the inner tube the release ring needs to be lifted with the thumb. When removing the thumb from the ring the inner tube is automatically fixed at the requested position.

2.) **Please do hold the upper tube**, when releasing the adjustment screw for adjusting the tube. Position the tube into the required height and do tighten the adjustment screw. Do not release the upper tube until the clamping force is sufficient for the requested use. Please do not forget to include the additional weight of the infusion bottles.

**⚠️ Attention!** Do not open the screw for height adjustment without holding the inner tube because the load might slide down uncontrolled. Fully loaded IV-hooks 8 kg / 17,6 lbs!

**⚠️ Attention!** Potential Eye Hazard. Maintain a safe distance from IV-hooks!

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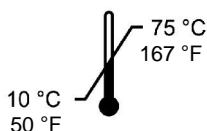
rF 30-75%  
700-1060 hPa

## 5. Cleaning and maintenance

For cleansing and disinfection, we recommend a disinfectant wipe with a disinfectant surface.

**!** Follow the instructions for cleaning and disinfection of the manufacturer as well as the current valid health regulations.

**!** None pH-neutral and chemically active disinfectants can lead to surface and material damage in dependency with exposure time and “aggressiveness” (pH, chemical reactivity, aggressiveness of solvents).

The following products / ingredients should not be used when cleaning and disinfecting:

- hydrocarbon solvents (benzine, thinner, ...) can damage coatings and plastics.
- agents with more than 10% alcohol content for coated / plastic parts
- without strong acids or alkalis
- Halide or Halogen releasing compounds (chlorine, bromine, iodine; these can also cause corrosion in stainless steel)
- Surface-scratching products (e.g. abrasive, wire brush, steel wool)
- Iron particles containing water / cleaning sponges
- Hydrochloric acid containing products

**!** **WARNING!**

**Do not apply any mechanical cleaning and disinfection procedures (washer system, pressure washer)**

### Inspection

The infusion stand should be checked at least once a year for proper functionality of all important components:

- Clamp-function for the inner tube still adequate.
- To assure functionality of the locking and antistatic rollers clean them up.
- Are all screws still in a safe and strong position.

Repairs have to be done by authorized personal or customer service with original equipment only.

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