


Instructions Digiwrap® finger/toe

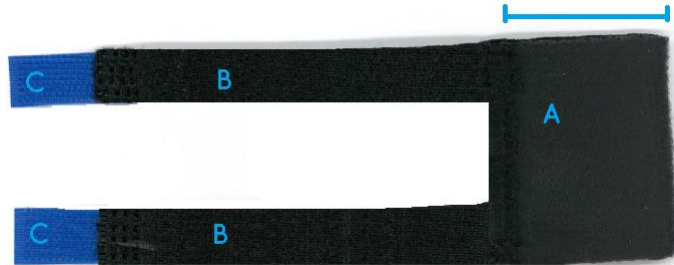

Size chart:

Article number	Size	Color	Dimensions	Package content
SP 10321	1	Yellow	< 6 cm	12 pcs
SP 10322	2	Red	6 - 6,5 cm	12 pcs
SP 10323	3	White	6,5 - 7 cm	12 pcs
SP 10324	4	Blue	7 - 7,5 cm	12 pcs
SP 10325	5	Black	7,5 - 8 cm	12 pcs
SP 10326	6	Green	8 - 9 cm	12 pcs
SP 10327	1 till 6		6 - 9 cm	6 pcs per size

Instructions for using Digiwrap® finger or toe.

Measure around the affected PIP-joint and see which size Digiwrap® is needed.

1. Slide cuff (A) over the injured finger or toe.
2. Wrap the two long sections (B) around the middle adjacent (NOT wounded) finger or toe.
3. Then wrap the two long sections around the injured finger or toe.
4. Stick the Velcro (C) distally onto the cuff.

Thus support and stability is given to the affected finger or toe.

Warning:

Check that the affected finger/toe does not discolour after the Digiwrap® has been applied and that no swelling occurs.

If the pain, swelling or stiffness increases or skin rash occurs, please contact your doctor.

Cleaning instruction:



Handwash Only



Do Not Tumble Dry



Do Not Iron



Do Not Bleach

